

## Recommended protocol for TdT

Obtained in run 52

03 Jan 2018

### Immunostainer

Type: Dako Omnis

### Primary antibody

Clone: EP266  
Producer: Dako/Agilent  
Product no. / lot no.: M3651 / 10116861  
Diluent: Antibody Diluent  
Dilution factor: 1:25  
Incubation time / temperature: 20 min. / 21°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 21°C

### Chromogen

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 21°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.