

## Recommended protocol for VIM

Obtained in run 52

04 Dec 2017

**Immunostainer**

Type: Dako Autostainer Link 48 +

**Primary antibody**

Clone: V9  
Producer: Dako/Agilent  
Product no. / lot no.: M0725 / 20027462  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:500  
Incubation time / temperature: 30 min. / 21°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 91°C

**Visualization system**

Producer: Dako  
Product / no: EnVision FLEX+ / K8012/DM822  
Linker: Linker, Mouse  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 21°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 21°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.