

## Recommended protocol for PAX5

Obtained in run 53

16 Feb 2018

**Immunostainer**

Type: DBS Montage 360 system

**Primary antibody**

Clone: BV6  
Producer: DBS  
Product no. / lot no.: RMAB027 / I139  
Diluent: Primary Antibody Diluent  
Diluent producer / no.: DBS / K004  
Dilution factor: 1:10  
Incubation time / temperature: 30 min. / 20°C

**Epitope retrieval, HIER**

Device: Pressure Cooker  
Buffer: DBS DBS Montage Citrate Antigen Retrieval Solution  
Heating time at max. temp.: 15 min.  
Maximum heating temp.: 105°C

**Visualization system**

Producer: DBS  
Product / no: Montage PolyVue Plus Auto Detection System / PVP100-A  
Linker: Mouse and Rabbit Link  
Incubation time linker: 10 min.  
Incubation time polymer: 10 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: DBS  
Product / no: Stable DAB/Plus / K047  
Incubation time / temperature: 5 min. / 20°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.