

## Recommended protocol for MUM1

Obtained in run 48

19 Aug 2016

**Immunostainer**

Type: Gene Tech Genestainer

**Primary antibody**

Clone: MUM1p  
Producer: Genemed  
Product no. / lot no.: 61-0136 / 60425211  
Diluent: Antibody diluent  
Diluent producer / no.: Gene Tech / GT100910  
Dilution factor: 1:100  
Incubation time / temperature: 50 min. / 25°C

**Epitope retrieval, HIER**

Device: Pressure Cooker  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 3 min.  
Maximum heating temp.: 110°C

**Visualization system**

Producer: Gene Tech  
Product / no: GTVision / GK6005  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

**Chromogen**

Producer: Gene Tech  
Product / no: DAB Detection Kit / GK6005  
Incubation time / temperature: 5 min. / 25°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.