

## Recommended protocol for OCT3/4

Obtained in run 53

02 Mar 2018

**Immunostainer**

Type: Biocare medical Intellipath

**Primary antibody**

Clone: N1NK  
Producer: Leicabiosystems  
Product no. / lot no.: NCL-L-OCT3/4 / 6050128  
Diluent: Davinci green  
Diluent producer / no.: Biocare medical / PD900M  
Dilution factor: 1:50  
Incubation time / temperature: 30 min. / 20°C

**Epitope retrieval, HIER**

Device: Pressure Cooker  
Buffer: Biocare medical Diva  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 110°C

**Visualization system**

Producer: Biocare medical  
Product / no: Mach4 Universal HRP-polymer / BRI4012L  
Linker: None  
Incubation time polymer: 10 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Biocare medical  
Product / no: Mach4 universal DAB / BRI4012L  
Incubation time / temperature: 5 min. / 20°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.