

## Recommended protocol for OCT3/4

Obtained in run 53

14 Mar 2018

**Immunostainer**

Type: Dako Omnis

**Primary antibody**

Clone: N1NK  
Producer: Dako/Agilent  
Product no. / lot no.: M3649 / 10134329  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S2022  
Dilution factor: 1:50  
Incubation time / temperature: 20 min. / 25°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

**Chromogen**

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 25°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.