

## Recommended protocol for ER

Obtained in run B26

27 Aug 2018

### Immunostainer

Type: Thermo Autostainer 36/48/72

### Primary antibody

Clone: 6F11  
Producer: Leica/novocastra  
Product no. / lot no.: NCL-L-ER-6F11 / 6058858  
Diluent: Bond Antibody Diluent  
Dilution factor: 1:50  
Incubation time / temperature: 50 min. / 25°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Novocastra, Leica epitope retrieval solution  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Novolink Max Polymer  
Product / no: Novolink Polymer Detection System / RE72060-CE  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Leica biosystems  
Product / no: DAB chromogen / RE7162  
Incubation time / temperature: 5 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.