

## Recommended protocol for WT1

Obtained in run 55

10 Dec 2018

**Immunostainer**

Type: Thermo Autostainer 36/48/72

**Primary antibody**

Clone: WT49  
Producer: Leica/Novocastra  
Product no. / lot no.: NCL-L-WT1-562 / 6062869  
Diluent: normal antibody diluent  
Diluent producer / no.: Immunologic / VWRKBD09-999  
Dilution factor: 1:10  
Incubation time / temperature: 60 min. / 20°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 10 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako  
Product / no: EnVision FLEX+ / K8002/SM802  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.