

Recommended protocol for CK5

Obtained in run 55

14 Dec 2018

Immunostainer

Type: Manual

Primary antibody

Clone: XM26/LL002
Producer: Zytomed Systems
Product no. / lot no.: MSK106 / V925
Diluent: Antibody Diluent
Diluent producer / no.: Zytomed Systems / ZUC025
Dilution factor: 1:50
Incubation time / temperature: 60 min. / 22°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Zytomed-Systems HIER-Citrate
Heating time at max. temp.: 7 min.
Maximum heating temp.: 110°C

Visualization system

Producer: Zytomed-Systems
Product / no: HRP-Polymer / POLHRP-100
Linker: Postblock
Incubation time linker: 20 min.
Incubation time polymer: 30 min.
Incubation temperature: 22°C

Chromogen

Producer: Zytomed Systems
Product / no: DAB high contrast / DAB500plus
Incubation time / temperature: 10 min. / 22°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.