

## Recommended protocol for CK5

Obtained in run 55

21 Dec 2018

### Immunostainer

Type: Gene Tech Genestainer

### Primary antibody

Clone: XM26  
Producer: Leica/novocastra  
Product no. / lot no.: NCL-L-CK5 / NCL-L-CK5  
Diluent: Ab diluent  
Diluent producer / no.: Gene Tech / GT100910  
Dilution factor: 1:100  
Incubation time / temperature: 50 min. / 25°C

### Epitope retrieval, HIER

Device: Water bath  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 15 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Gene Tech  
Product / no: GTVision / GK6007  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Gene Tech  
Product / no: DAB Detection Kit / GK6007  
Incubation time / temperature: 5 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.