

## Recommended protocol for EpCAM

Obtained in run 56

04 Feb 2019

**Immunostainer**

Type: Dako Omnis

**Primary antibody**

Clone: VU-1D9  
Producer: Thermo Scientific  
Product no. / lot no.: MS-144-P / 144P1801G  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:400  
Incubation time / temperature: 20 min. / 32°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, Low pH  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 32°C

**Chromogen**

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 32°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.