

## Recommended protocol for PD-L1 (lung)

Obtained in run C5

05 Feb 2019

**Immunostainer**

Type: Dako Autostainer Link 48 +

**Primary antibody**

Clone: E1L3N  
Producer: Cell Signaling Technology  
Product no. / lot no.: 13684 / 7  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S3022  
Dilution factor: 1:150  
Incubation time / temperature: 40 min. / 22°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Dako TRS Low pH (3-1)  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 95°C

**Visualization system**

Producer: Dako  
Product / no: EnVision FLEX / K8000/SM802  
Linker: Linker, Rabbit  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 22°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 22°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.