

## Recommended protocol for MLH1

Obtained in run 56

03 May 2019

**Immunostainer**

Type: Dako Omnis

**Primary antibody**

Clone: GM011  
Producer: Biosite  
Product no. / lot no.: 254-EBCB69 / 80479205  
Diluent: Renoir Red Diluent  
Diluent producer / no.: Biocare Medical / PD904  
Dilution factor: 1:300  
Incubation time / temperature: 30 min. / 32°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: Mouse LINKER  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 32°C

**Chromogen**

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 32°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.