

Recommended protocol for Bcl-2

Obtained in run 57

02 Aug 2019

Immunostainer

Type: Lumatas Autostaining System Mo

Primary antibody

Clone: MX022
Producer: maixin
Product no. / lot no.: MAB-0711 / 1901090711a
Diluent: None
Diluent producer / no.: None / None
Dilution factor: 1:0
Incubation time / temperature: 30 min. / 32°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 99°C

Visualization system

Producer: maixin
Product / no: DAB (Titan super) / TT-0805
Incubation time polymer: 13 min.
Amplifier: DAB (Titan super)
Incubation time amplifier: 13 min.
Incubation temperature: 25°C

Chromogen

Producer: maixin
Product / no: DAB (Titan super) / TT-0805
Incubation time / temperature: 6 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.