

Recommended protocol for Ki67

Obtained in run B22

02 Sep 2016

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: BS4
Producer: Nordic Biosite
Product no. / lot no.: BSH-7302 / BSH1-2aa
Diluent: Antibody diluent
Dilution factor: 1:200
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Nordic Biosite
Product / no: Polymer anti mouse HRP / KDB-10007
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Nordic Biosite
Product / no: High contrast DAB / BCB-20032
Incubation time / temperature: 10 min. / 20°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.