

Recommended protocol for HER2 IHC

Obtained in run B28

27 Aug 2019

Immunostainer

Type: Leica BOND Max

Primary antibody

Clone: SP3
Producer: Thermo Scientific
Product no. / lot no.: RM-9103-S-A / 9103S1807B
Diluent: Bond Antibody Diluent
Dilution factor: 1:100
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica Bond Epitope Retrieval Solution 2
Heating time at max. temp.: 20 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 20°C

Chromogen

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.