

## Recommended protocol for MSH2

Obtained in run 57

27 Aug 2019

**Immunostainer**

Type: Leica BOND III

**Primary antibody**

Clone: RED2  
Producer: Epitomics  
Product no. / lot no.: AC-0211 / EP040503  
Diluent: Bond Antibody Diluent  
Diluent producer / no.: Leica / AR9352  
Dilution factor: 1:800  
Incubation time / temperature: 60 min. / 22°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Leica Bond Epitope Retrieval Solution 2  
Heating time at max. temp.: 40 min.  
Maximum heating temp.: 99°C

**Visualization system**

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 15 min.  
Incubation time polymer: 15 min.  
Incubation temperature: 22°C

**Chromogen**

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 10 min. / 22°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.