

## Recommended protocol for SATB2

Obtained in run 58

02 Jan 2020

**Immunostainer**

Type: Ventana Benchmark Ultra

**Primary antibody**

Clone: SP281  
Producer: Abcam  
Product no. / lot no.: ab227686 / ab227686  
Diluent: Antibody Diluent  
Diluent producer / no.: Ventana / 251-018  
Dilution factor: 1:250  
Incubation time / temperature: 32 min. / 37°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 32 min.  
Maximum heating temp.: 100°C

**Visualization system**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 36°C

**Chromogen**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.