

## Recommended protocol for MUM1

Obtained in run 58

06 Jan 2020

**Immunostainer**

Type: Leica BOND III

**Primary antibody**

Clone: EAU32  
Producer: Leica Biosystems  
Product no. / lot no.: MUM1-L-CE / 6068445  
Diluent: Bond Antibody Diluent  
Diluent producer / no.: Leica / AR9352  
Dilution factor: 1:300  
Incubation time / temperature: 12 min. / 36°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Leica Bond Epitope Retrieval Solution 1  
Heating time at max. temp.: 40 min.  
Maximum heating temp.: 36°C

**Visualization system**

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 4 min.  
Incubation time polymer: 4 min.  
Incubation temperature: 36°C

**Chromogen**

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.