

Recommended protocol for p57

Obtained in run

23 Sep 2015

Immunostainer

Type: Freedom Evo, Teecan

Primary antibody

Clone: 57P06 (KP10)
Producer: Thermo Scientific
Product no. / lot no.: MS-1062-P / 1062P1310C
Diluent: Antibody Diluent
Diluent producer / no.: NS / NS
Dilution factor: 1:2000
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Citrate pH 6
Heating time at max. temp.: 30 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Immunologic
Product / no: NS / DPVO-999HRPN
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / K3467 / K3468
Incubation time / temperature: 5 min. / 20°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.