

## Recommended protocol for S100

Obtained in run 59

07 Feb 2020

**Immunostainer**

Type: Dako Omnis

**Primary antibody**

Clone: Polyclonal  
Producer: Dako  
Product no. / lot no.: Z0311 / 00080275  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:5000  
Incubation time / temperature: 30 min. / 32°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 24 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: Rabbit LINKER  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 32°C

**Chromogen**

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 10 min. / 32°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.