

Recommended protocol for CD45

Obtained in run 59

13 Feb 2020

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: 2B11+PD7/26

Producer: Dako

Product no. / lot no.: M0701 / 20064344

Diluent: Antibody Diluent

Diluent producer / no.: Dako / K8006

Dilution factor: 1:100

Incubation time / temperature: 20 min. / 32°C

Epitope retrieval, HIER

Device: On Board / On Machine

Buffer: Dako Omnis Target Retrieval Solution, High pH

Heating time at max. temp.: 30 min.

Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis

Product / no: EnVision Flex / GV800/GV823

Linker: None

Incubation time polymer: 20 min.

Incubation temperature: 32°C

Chromogen

Producer: Dako Omnis

Product / no: DAB+ Substrate Chromogen System / GV825

Incubation time / temperature: 5 min. / 32°C

Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.