

## Recommended protocol for CD68

Obtained in run 60

03 Aug 2020

**Immunostainer**

Type: Ventana Benchmark Ultra

**Primary antibody**

Clone: KP1  
Producer: Ventana Roche  
Product no. / lot no.: 790-2931 / E29406  
Format: Ready-To-Use (prediluted)  
Incubation time / temperature: 12 min. / 36°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 40 min.  
Maximum heating temp.: 95°C

**Visualization system**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Amplifier: OptiView Amplification Kit  
Incubation time amplifier: 4+4 min.  
Incubation temperature: 36°C

**Chromogen**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.