

Recommended protocol for PAX8

Obtained in run 60

20 Aug 2020

Immunostainer

Type: Dako Autostainer Link 48 +

Primary antibody

Clone: BP6157
Producer: Bailing Biotechnology Co., Ltd
Product no. / lot no.: BX50179 / CPL701203P
Diluent: Tris-HCl with BSA (pH7.2-7.4)
Diluent producer / no.: Biolynx //
Dilution factor: 1:800
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 30 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX+ / K8002/SM802
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.