

Recommended protocol for MLA

Obtained in run 60

19 Oct 2020

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: BS52
Producer: Nordic Biosite
Product no. / lot no.: BSH-2003-1 / BSH1-7ac
Diluent: Antibody Diluent
Diluent producer / no.: BioSite Histo / BCB-QUG2XK-500
Dilution factor: 1:100
Incubation time / temperature: 30 min. / 21°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 98°C

Visualization system

Producer: BioSite Histo
Product / no: (HRP) Polymer anti-Mouse / KDB-OZRNGT-100
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 21°C

Chromogen

Producer: BioSite Histo
Product / no: DAB Substrate Buffer / BCB-R7IKBJ-1
Incubation time / temperature: 10 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.