

## Recommended protocol for CD163

Obtained in run

24 Sep 2015

**Immunostainer**

Type: Ventana Benchmark Ultra

**Primary antibody**

Clone: MRQ-26  
Producer: Cell Marque  
Product no. / lot no.: 163M-16 / 6119  
Diluent: 0.025M TBS pH 7.4  
Diluent producer / no.: /  
Dilution factor: 1:40  
Incubation time / temperature: 32 min. / 36°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 99°C

**Visualization system**

Producer: Ventana  
Product / no: UltraView Universal DAB Detection Kit / 760-500  
Incubation time polymer: 8 min.  
Incubation temperature: 36°C

**Chromogen**

Producer: Ventana  
Product / no: ultraView Universa DAB Detection Kit / 760-500  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.