

## Recommended protocol for CD56

Obtained in run 61

04 Feb 2021

**Immunostainer**

Type: Ventana Benchmark Ultra

**Primary antibody**

Clone: CD564  
Producer: Leica/novocastra  
Product no. / lot no.: NCL-CD56-504 / 6061098  
Diluent: Antibody Diluent with Casein  
Diluent producer / no.: Ventana / 760-219  
Dilution factor: 1:100  
Incubation time / temperature: 120 min. / 37°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 100°C

**Visualization system**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 21°C

**Chromogen**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 21°C  
Enhancement: CuSO4

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.