

Recommended protocol for CDX2

Obtained in run 61

05 Feb 2021

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: EPR2764Y
Producer: Cell Marque
Product no. / lot no.: 235R-14, -15 or -16 / 64990
Diluent: Antibody Diluent
Diluent producer / no.: Dako / S2022
Dilution factor: 1:200
Incubation time / temperature: 20 min. / 32°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 30 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823
Linker: Rabbit LINKER
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Incubation temperature: 32°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 32°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.