

## Recommended protocol for ER

Obtained in run B31

18 Dec 2020

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: 6F11  
Producer: Leica/novocastra  
Product no. / lot no.: NCL-L-ER-6F11 / 6072770  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S0809  
Dilution factor: 1:50  
Incubation time / temperature: 24 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 32 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.