

## Recommended protocol for CD56

Obtained in run 61

21 Dec 2020

**Immunostainer**

Type: Leica BOND III

**Primary antibody**

Clone: CD56 (CD564) RTU

Producer: Leica

Product no. / lot no.: PA0191 / 67920

Diluent: None

Diluent producer / no.: None / None

Dilution factor: 1:0

Incubation time / temperature: 15 min. / 25°C

**Epitope retrieval, HIER**

Device: On Board / On Machine

Buffer: Leica Bond Epitope Retrieval Solution 2

Heating time at max. temp.: 20 min.

Maximum heating temp.: 100°C

**Visualization system**

Producer: Leica

Product / no: Bond Refine / DS9800

Incubation time linker: 8 min.

Incubation time polymer: 8 min.

Incubation temperature: 25°C

**Chromogen**

Producer: Leica

Product / no: Bond Refine / DS9800

Incubation time / temperature: 10 min. / 25°C

Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.