

## Recommended protocol for CDX2

Obtained in run 61

05 Jan 2021

**Immunostainer**

Type: Dako Omnis

**Primary antibody**

Clone: EP25  
Producer: Epitomics  
Product no. / lot no.: AC-0008 / 19051401  
Diluent: Renoir Red Diluent  
Diluent producer / no.: Biocare Medical / PD904  
Dilution factor: 1:200  
Incubation time / temperature: 30 min. / 32°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 24 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: Rabbit LINKER  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 32°C

**Chromogen**

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 10 min. / 32°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.