

## Recommended protocol for CD31

Obtained in run 62

05 Mar 2021

**Immunostainer**

Type: Tissue-Tek Genie

**Primary antibody**

Clone: RM247  
Producer: SAKURA  
Product no. / lot no.: 8282-C010 / 8282A28320  
Diluent: None  
Diluent producer / no.: None / None  
Dilution factor: 1:0  
Incubation time / temperature: 30 min. / 25°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: SAKURA Tissue-Tek High pH  
Heating time at max. temp.: 45 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: SAKURA  
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250  
Incubation time linker: 15 min.  
Incubation time polymer: 10 min.  
Incubation temperature: 25°C

**Chromogen**

Producer: 8826-K250  
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / SAKURA  
Incubation time / temperature: 8 min. / 25°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.