

## Recommended protocol for CK20

Obtained in run 62

15 Mar 2021

**Immunostainer**

Type: Sakura Genie

**Primary antibody**

Clone: Ks 20.8  
Producer: Sakura  
Product no. / lot no.: 8304 / 8304A34620  
Diluent: None  
Diluent producer / no.: None / None  
Dilution factor: 1:0  
Incubation time / temperature: 30 min. / 21°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Sakura Sakura Finetek USA High pH Antigen Retrieval solution  
Heating time at max. temp.: 45 min.  
Maximum heating temp.: 98°C

**Visualization system**

Producer: Sakura  
Product / no: Tissue-Tek Genie Pro kit / 8826-K250  
Incubation time polymer: 20 min.  
Incubation temperature: 21°C

**Chromogen**

Producer: Sakura  
Product / no: DAB / 8826-K250  
Incubation time / temperature: 6 min. / 21°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.