

Recommended protocol for CK20

Obtained in run 62

23 Apr 2021

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: BS101
Producer: Nordic Biosite
Product no. / lot no.: BSH-2000-1 / BSH1-4s
Diluent: Antibody Diluent
Diluent producer / no.: Nordic BioSite / BCB-QUG2XK-500
Dilution factor: 1:100
Incubation time / temperature: 30 min. / 21°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Nordic BioSite
Product / no: (HRP) Polymer anti-Mouse / KDB-OZRNGT-100
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 21°C

Chromogen

Producer: Nordic BioSite
Product / no: DAB High Contrast Kit / BCB-R7IKBJ-1
Incubation time / temperature: 10 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.