

Recommended protocol for VIM

Obtained in run

26 Sep 2015

Immunostainer

Type: Dako Autostainer

Primary antibody

Clone: V9
Producer: Dako
Product no. / lot no.: M0725 / 00038159
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:400
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH
Heating time at max. temp.: 20 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX+ / K8012/DM822
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / K3467 / K3468
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.