

## Recommended protocol for EMA

Obtained in run

26 Sep 2015

**Immunostainer**

Type: Dako Autostainer

**Primary antibody**

Clone: E29  
Producer: Dako  
Product no. / lot no.: M0613 / 00041785  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S0809  
Dilution factor: 1:500  
Incubation time / temperature: 30 min. / 20°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako  
Product / no: REAL EnVision Rabbit/Mouse / K5007 / K4065  
Incubation time polymer: 30 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / K5007 / K4065  
Incubation time / temperature: 7 min. / 20°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.