

## Recommended protocol for MSA

Obtained in run

26 Sep 2015

### Immunostainer

Type: IntelliPath, Biocare

### Primary antibody

Clone: HMB-45  
Producer: Dako  
Product no. / lot no.: M0634 / 1075695  
Diluent: Antibody Diluent  
Diluent producer / no.: Nordic Biosite / NS  
Dilution factor: 1:50  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: Pressure Cooker  
Buffer: Biocare Diva Diloaker pH 6,2  
Heating time at max. temp.: 5 min.  
Maximum heating temp.: 110°C

### Visualization system

Producer: Biocare  
Product / no: NS / M4U534L  
Linker: NS  
Incubation time linker: 10 min.  
Incubation time polymer: 10 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Biocare  
Product / no: DAB / M4U534L  
Incubation time / temperature: 5 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.