

## Recommended protocol for CK-HMW

Obtained in run

26 Sep 2015

**Immunostainer**

Type: Autostainer, LabVision

**Primary antibody**

Clone: LL002  
Producer: Leica/novocastra  
Product no. / lot no.: NCL-LL002 / 6016952  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:10  
Incubation time / temperature: 30 min. / 20°C

**Epitope retrieval, HIER**

Device: Microwave oven  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 10 min.  
Maximum heating temp.: 100°C

**Visualization system**

Producer: Dako  
Product / no: Real EnVision Rabbit/Mouse / K5007  
Incubation time polymer: min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / K5007 / K4065  
Incubation time / temperature: 14 min. / 20°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.