

## Recommended protocol for CK-LMW

Obtained in run

26 Sep 2015

**Immunostainer**

Type: Autostainer, LabVision

**Primary antibody**

Clone: C51  
Producer: Invitrogen / Zymed  
Product no. / lot no.: 18-185Z / 762348A  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S0809  
Dilution factor: 1:50  
Incubation time / temperature: 30 min. / 20°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 98°C

**Visualization system**

Producer: Dako  
Product / no: EnVision+ System- HRP / K4001  
Incubation time polymer: 30 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / K3467 / K3468  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.