

Recommended protocol for DES

Obtained in run 64

03 Feb 2022

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: DE-R-11
Producer: Ventana
Product no. / lot no.: 760-2513 / H04767
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 24 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 24 min.
Maximum heating temp.: 100°C

Epitope retrieval, proteolysis

Enzyme: Protease 3
Enzyme producer / no: Ventana / 760-2020
Incubation time / temp: 4 min. / 36°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided



Nordic Immunohistochemical Quality Control

Institute of Pathology, Aalborg University Hospital, Ladegaardsgade 3, P.O.Box 561, DK-9100 Aalborg, Denmark

information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.