

## Recommended protocol for CR

Obtained in run 64

12 Jan 2022

### Immunostainer

Type: CNT330

### Primary antibody

Clone: C5G4  
Producer: Celnovte  
Product no. / lot no.: CCM-0222 / 202112001  
Diluent: None  
Diluent producer / no.: None / None  
Dilution factor: 1:0  
Incubation time / temperature: 30 min. / 25°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 100°C

### Visualization system

Producer: Celnovte  
Product / no: CNTVision Super PolyDetection / SD5300  
Linker: CNTVision Super PolyDetection  
Incubation time linker: 20 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Celnovte  
Product / no: CNTVision Super PolyDetection / SD5300  
Incubation time / temperature: 6 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.