

## Recommended protocol for CD56

Obtained in run 64

22 Feb 2022

**Immunostainer**

Type: Tissue-Tek Genie Advanced Stai

**Primary antibody**

Clone: MRQ-42  
Producer: Sakura Finetek  
Product no. / lot no.: 8274-C010 / 8274A13121  
Format: Ready-To-Use (prediluted)  
Incubation time / temperature: 30 min. / 20°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Sakura Finetek Tissue-Tek Genie High pH Antigen Retrieval Buffer  
Heating time at max. temp.: 45 min.  
Maximum heating temp.: 98°C

**Visualization system**

Producer: Sakura Finetek  
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250  
Linker: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Sakura Finetek  
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250 / 8826-K250  
Incubation time / temperature: 6 min. / 25°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.