

Recommended protocol for CR

Obtained in run 64

03 Jan 2022

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: BSR235
Producer: NordicBiosite
Product no. / lot no.: BSH-4005 / BSH1-8cf
Diluent: Antibody Diluent
Diluent producer / no.: ZYTOMED systems / ZUC025-500
Dilution factor: 1:100
Incubation time / temperature: 30 min. / 21°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 98°C

Visualization system

Producer: ZYTOMED systems
Product / no: (HRP) Polymer anti-Rabbit / ZUC032-100
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 21°C

Chromogen

Producer: ZYTOMED systems
Product / no: DAB substrate High contrast kit / DAB5000PLUS
Incubation time / temperature: 10 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.