

## Recommended protocol for CD56

Obtained in run 64

10 Jan 2022

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: 1B6  
Producer: Leica/novocastra  
Product no. / lot no.: NCL-CD56-1B6 / 6089053  
Diluent: Antibody Diluent  
Diluent producer / no.: Ventana / 251-018  
Dilution factor: 1:100  
Incubation time / temperature: 48 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 92 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 12 min.  
Incubation time polymer: 12 min.  
Amplifier: OptiView Amplification Kit  
Incubation time amplifier: 12+12 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 12 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect



## **Nordic Immunohistochemical Quality Control**

Department of Pathology, Aalborg University Hospital, Hospitalsbyen 5, DK-9260 Gistrup, Denmark

the most recent developments in immunohistochemistry at any point in time.