

## Recommended protocol for CD56

Obtained in run 64

12 Jan 2022

**Immunostainer**

Type: Thermo Autostainer 36/48/72

**Primary antibody**

Clone: 123C3  
Producer: VITRO SA  
Product no. / lot no.: MAD-000749QD / 07490023S  
Diluent: None  
Diluent producer / no.: None / None  
Dilution factor: 1:1  
Incubation time / temperature: 10 min. / 23°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Std EDTA / EGTA pH 8  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 95°C

**Visualization system**

Producer: VITRO SA  
Product / no: MASTER POLYMER PLUS / MAD-000237QD  
Linker: None  
Incubation time polymer: 15 min.  
Incubation temperature: 23°C

**Chromogen**

Producer: VITRO SA  
Product / no: DAB / MAD-001816QK  
Incubation time / temperature: 5 min. / 23°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.