

## Recommended protocol for Napsin A

Obtained in run 66

04 Aug 2022

**Immunostainer**

Type: Dako Autostainer Link 48 +

**Primary antibody**

Clone: TMU-Ad02  
Producer: BioCare  
Product no. / lot no.: CM388CK / 111620  
Diluent: Renoir Red Diluent  
Diluent producer / no.: Biocare Medical / PD904  
Dilution factor: 1:100  
Incubation time / temperature: 20 min. / 22°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako  
Product / no: EnVision FLEX+ / K8002/SM802  
Linker: Linker, Mouse  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 22°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 22°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.