

Recommended protocol for Napsin A

Obtained in run 66

22 Aug 2022

Immunostainer

Type: Gene Tech GeneStainer

Primary antibody

Clone: IP64
Producer: Leica/novocastra
Product no. / lot no.: NCL-L-Napsin A / 6095005
Diluent: Ab diluent
Diluent producer / no.: Gene Tech / GT100910
Dilution factor: 1:100
Incubation time / temperature: 50 min. / 25°C

Epitope retrieval, HIER

Device: Water bath
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 95°C

Visualization system

Producer: Gene Tech
Product / no: GTVision / GK8007
Linker: None
Incubation time polymer: 20 min.
Incubation temperature: 25°C

Chromogen

Producer: Gene Tech
Product / no: DAB / GK8007
Incubation time / temperature: 5 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.