

Recommended protocol for CK-LMW

Obtained in run 49

28 Dec 2016

Immunostainer

Type: Manual

Primary antibody

Clone: EP17
Producer: Epitomics
Product no. / lot no.: AC-0007 / EM120405
Diluent: Antibody Diluent
Diluent producer / no.: Dako / S2022
Dilution factor: 1:100
Incubation time / temperature: 30 min. / 25°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Diagnostic Bio Syste Tris-EDTA RB pH9,0
Heating time at max. temp.: 30 min.
Maximum heating temp.: 96°C

Visualization system

Producer: Dako
Product / no: REAL EnVision Rabbit/Mouse / K5007 / K4065
Incubation time polymer: 30 min.
Incubation temperature: 25°C

Chromogen

Producer: Dako
Product / no: DAB+ / K5007 / K4065
Incubation time / temperature: 5 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.