

## Recommended protocol for p40

Obtained in run 67

28 Dec 2022

**Immunostainer**

Type: Ventana Benchmark Ultra

**Primary antibody**

Clone: ZR8  
Producer: Immunologic  
Product no. / lot no.: ILM0022-C1, -C05 or / 310322  
Diluent: Antibody Diluent  
Diluent producer / no.: Ventana / 251-018  
Dilution factor: 1:50  
Incubation time / temperature: 32 min. / 36°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 40 min.  
Maximum heating temp.: 100°C

**Visualization system**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 20°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.