

## Recommended protocol for CD4

Obtained in run 67

29 Dec 2022

### Immunostainer

Type: Gene Tech GeneStainer

### Primary antibody

Clone: EP204  
Producer: Epitomics  
Product no. / lot no.: AC-0173 / 230100969  
Diluent: Ab diluent  
Diluent producer / no.: Gene Tech / GT100910  
Dilution factor: 1:150  
Incubation time / temperature: 50 min. / 25°C

### Epitope retrieval, HIER

Device: Water bath  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Gene Tech  
Product / no: GTVision / GK8007  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Gene Tech  
Product / no: DAB / GK8007  
Incubation time / temperature: 5 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.